

DERMIS PLUS PADS AND STRIPS®

SKIN INTEGRITY PRODUCTS

Ensuring Comfort and Care



Based on the policy that prevention is better than cure, OakMed products have introduced the OakMed Dermis Plus Gel Pad and Dermis Plus Gel Strip to assist with pressure offloading, in difficult to reach and treat areas.

DERMIS PLUS GEL PADS / DERMIS PLUS GEL STRIPS

Dermis Plus Gel Pads and Dermis Plus Gel Strips are recommended for pressure off-loading in patients that are medium to high risk of developing compromised skin integrity issues such as pressure ulcers, skin tears or calluses due to unrelieved pressure, shear and friction.

OakMed Dermis Plus Gel Pads and Dermis Plus Gel Strips are soft, conformable and elastic, and may be used for many different applications such as:

- Podiatry applications such as protection between toes, plantar fasciitis, and hallux valgus (bunions)
- Protection under breasts and over nipples
- Ear protection and bony cartilage
- Protection with orthopedic appliances such as splints and cervical collars
- Protection from collars, and splints, oxygen cannulas, nasal-gastric tubing and catheter tubing
- General operating theatre use to preserve and protect skin integrity
- Protection between skin folds and other vulnerable areas
- Protection of bony prominences such as elbows, shins and heels
- Use with prosthesis and other amputee issues, including contractions
- Protection of the sacral area
- Protection of the occipital area



Cushions
and Protects



Washable and
Reusable



Conforms



Mineral Oil
Inside




DERMIS PLUS PADS AND STRIPS®

SKIN INTEGRITY PRODUCTS

Ensuring Comfort and Care

DERMIS PLUS GEL PADS • SIZE INFORMATION

- **SMALL:** 1/10inch thickness, 4inch x 4inch sheets
- **LARGE:** 1/2inch thickness, 4inch x 4inch sheets

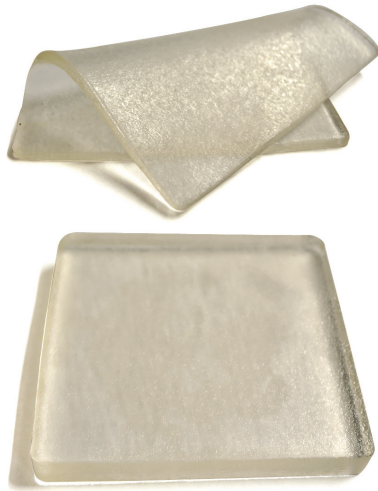

DERMIS PLUS GEL STRIPS • SIZE INFORMATION

- Sold in 3 foot lengths which may be cut to size
- Two styles available, tacky and soft



INSTRUCTIONS FOR USE

- Remove from box and where applicable, peel back the protective film
- Cut to size if required
- Apply to the affected area, use either side of the Dermis Plus product
- Cover with dry dressing and secure with tape or elastic garment if required
- May be washed with a mild soapy detergent - single patient use
- Use for no more than 16 hours maximum per day maximum and check regularly for any signs of abnormality
- If any skin irritation occurs remove immediately and consult a physician
- Do not autoclave or place in direct sunlight
- Diabetics and those with poor or compromised blood circulation should seek professional medical advice before and while using this product
- Keep out of the reach of children and pets, if swallowed seek immediate medical advice.
- **FOR EXTERNAL USE ONLY**
- **NOT TO BE USED AS A PRIMARY WOUND DRESSING OR OVER MOUTH OR NOSE**

KEY FEATURES AND BENEFITS

- Contains a cross-linked, three-dimensional polymer gel impregnated with a medical grade mineral oil that softens, moisturizes and lubricates the skin
- Greatly reduces shear and friction to the skin
- The gel flows in all directions when subjected to weight or pressure, therefore vibration, friction and shear forces are minimized
- High elasticity ensures a return to the original form after being stretched
- The polymer gel conforms easily to bony prominences providing superb comfort and primary pressure relief
- The gel has a 'memory' and will quickly return to its original form after being stretched or manipulated, and has an elasticity of 400%
- All products in the range are washable with mild soapy detergents and reusable
- Dermis Plus Gel Pads and Dermis Plus Gel Strips are gentle on the skin, hypoallergenic, non toxic and hygienic
- Dermis Plus Gel Pads and Dermis Plus Gel Strips may be cut and shaped to size according to need

